**MANUAL DE INSTRUCCIONES
DIRECTIONS FOR USE
NOTICE D'UTILISATION
MANUALE PER L'USO
MANUAL DO UTILIZADOR
GEBRAUCHSANLEITUNG**

**SUNRISE
MEDICAL**

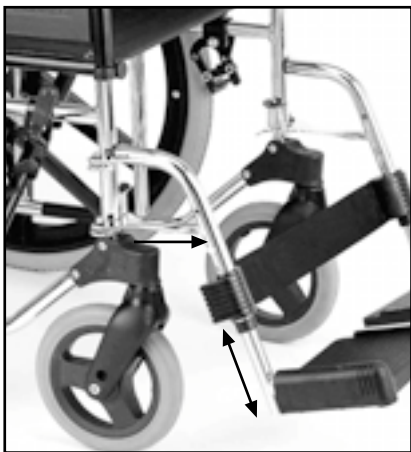
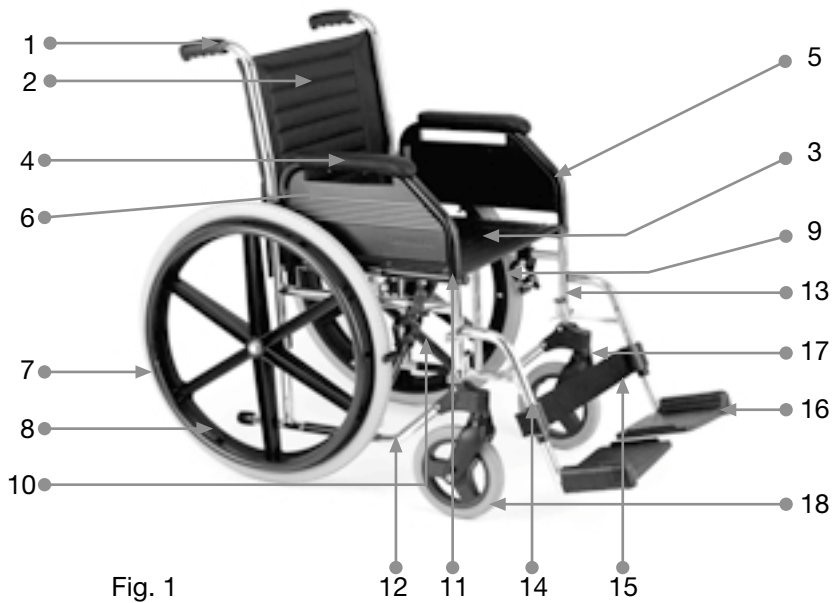
Fig. 2

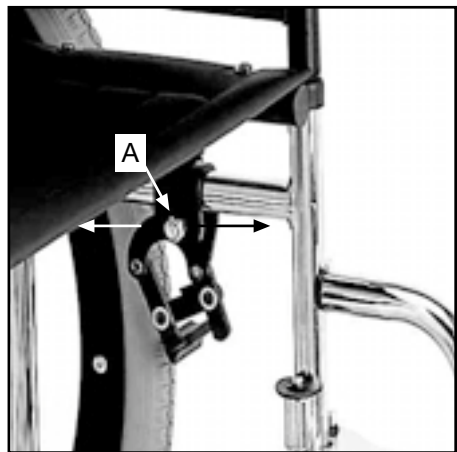
Fig. 3

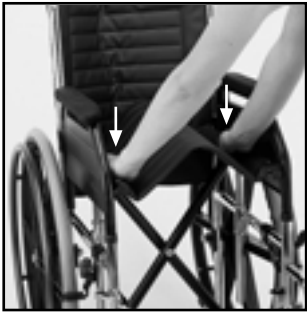
Fig. 4

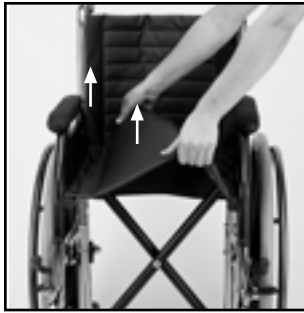
Fig. 5

Fig. 6

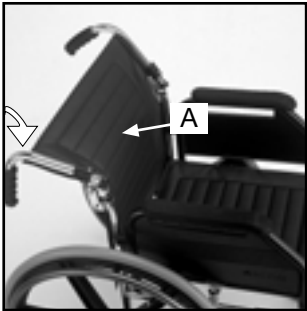
Fig. 7

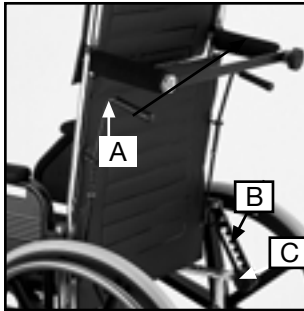
Fig. 8

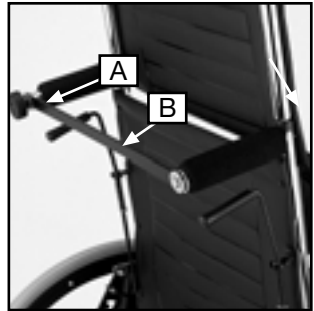
Fig. 9

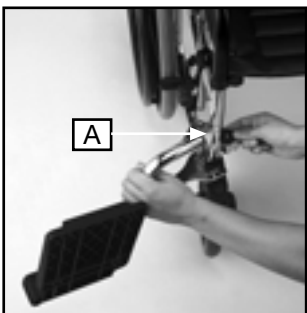
Fig. 10

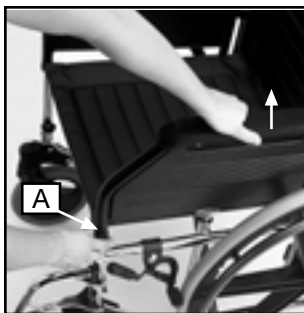
Fig. 11

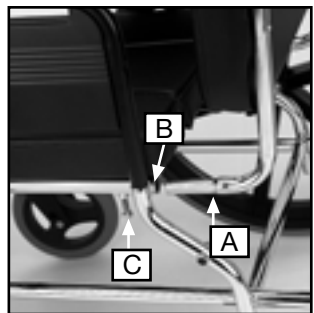
Fig. 12

INTRODUCCION

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INTRODUCTION

These wheelchairs have been designed for physically or mentally handicapped people with limited or no personal mobility only for transport.

Folding models are suitable for both indoor and outdoor use. Fixed models are suitable for indoor use

Folding wheelchairs: Refs.: 4007,4011, 4005,4012,4015,4020,4001,4021,4031, 4041,4010,4002.

Fixed wheelchairs: Refs.: 3004,3002,3003,3018,3013

SAFETY TIPS

The engineering and construction of this wheelchair have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must absolutely be observed. Unprofessional or incorrect adaptation or adjustment work could increase the risk of accidents. As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else. You should always wear clothing which is light-coloured when it is dark, so that you can be seen more easily. Be careful during your first ride in this wheelchair. Get to know your wheelchair. Practice using your wheelchair on level ground as well as going uphill and downhill.

CAUTION: The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheel locks to prevent such rolling. Always use both wheel locks; otherwise, your wheelchair could tip over. The wheel locks have not been designed to be used as brakes for a moving wheelchair.

Always make use of elevators and ramps. If these are not available, you can reach your destination with the aid of two attendants. They should grip the wheelchair at securely mounted parts. A wheelchair should never be lifted with an occupant; it should only be pushed.

Before each use, the following should be checked: .Quick-release axles (if fitted) on the rear wheels .Tyres, tyre pressure, and wheel locks. Wheel locks will not function properly if tire pressure is too low or the interval between tyres and wheel locks is too large.

The maximum user weight is 100 kilograms.

This wheelchair should be used exclusively to convey one person at a time in the seat. Any other use beyond this limit is in violation of safety regulations. Users should not sit in the wheelchair while riding in any and all means of transportation. When vehicles are in motion, unoccupied wheelchairs should be secured using appropriate means. It is possible that patholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill. When riding over a step or up an incline frontally, the body should be leaning forward.

FEATURES (Fig.1, pag.1)

- | | |
|-----------------------|----------------------|
| 1. Handgrip | 10. Cross brace |
| 2. Back upholstery | 11. Armrest release |
| 3. Seat upholstery | 12. Chassis |
| 4. Armrest upholstery | 13. Footrest release |
| 5. Armrest tube | 14. Footrest |
| 6. Armrest guard | 15. Calf strap |
| 7. Push rim | 16. Footplate |
| 8. Propulsion wheel | 17. Fork |
| 9. Wheel lock | 18. Castor |

HANDLING

1. OPENING AND FOLDING. (Fig 4, 5, Pag. 2)

Opening: Rest both hands on each of the seat upholstery support rails and press down until the seat and back upholsteries become tight. Lower the footplates into the using position. As a safety measure, the footplates should always be in an upright position before sitting or getting out of the chair in a forward movement.

Warning; Do not place your fingers between the seat upholstery and the seat tubes when opening or folding the wheelchair

Folding: Lift up the footplates. Grasp the centre of the seat upholstery from behind and in front, lifting the seat upwards. This will cause the chair to fold up by its own weight.

2. ELEVATING LEGRESTS (Std. in Mod.4001,4031) optional in other models (Ref. 21) (Fig.6 , pag.2)

Rotation of the legrest padding : If the padding is not to be used or the chair is to be folded, manually push the padding forward while turning it 90 degrees. When it is to be used once again, manually turn it towards the centre of the chair.

To lift: Press the black button and push the legrest upwards to the desired position.

To lower: Press the black button so that the leg rest can be lowered to the desired position.

3. BACKRESTS

3.1 Folding backrests (Ref. 4010 and 4015) (Fig 7, pag 2)

To fold the backrest, pull upwards the locking piece (A) and then pull downwards the back rest. To assemble: Pull upwards the back rest and pull downwards the plastic piece (A) to lock it and to secure it in this position.

3.2 Reclining backrest (Refs; 4001,4021,4031,4041,3013,3018) (fig. 8, pag.2)

Lowering the backrest: Loose the screw knobs (C) at the bottom of the racks. Lift the two handgrips (A) upwards at the same time, while keeping the chair steady by its handles. This movement releases the lower racks (B) from their housings leaving the backrest free to be reclined. Hold the backrest firmly to prevent it from dropping suddenly. If this should happen, quickly let go off the handgrips (A) so that the racks fall into place and prevent it from falling. With the handgrips (A) in an upward position, adjust the backrest to the desired angle using the racks' notches. Once this angle has been obtained, ensure that the racks have slipped into their notches (this operation should be done to prevent accidents caused by a sudden drop of the backrest). Release the handgrips(A). Tighten the screw at the bottom of the racks.

Warning: After the third rack (indicated with a red mark) the posterior stability of the chair is reduced, so proceed carefully.

Raising the backrest; Proceed in the same way as for reclining the back rest.

Warning: Before raising the backrest, be sure that the user's hands or arms are not between the seat and the reclined backrest, to avoid the risk of trapping them when raising the backrest.

3.3. Reclining backrests for folding chair with headrest (Mod 4001,4031) (fig.9, pag. 2)

Tensioning of backrest: Once the chair has been unfolded, place the strip (B) on the knob's screw (A) taking care that the strip's notch fits perfectly into the knob's screw. Manually tighten the knob (A). This tensioning operation should always be carried out before using the chair.

Untensioning of the backrest: Loosen the knob (A) by turning it several times but without removing it completely. Pull the strip (B) upwards. This operation must be carried out in order to fold the chair.

ASSEMBLY

1. LEGRESTS (FIG.10, PAG.2)

To remove: After having unfastened the calf strap from its fixture, lift the catch (A) and turn the right footrest towards the right and the left footrest towards the left. To remove the footrests from the chair, pull them upwards, after they have been turned, to release them from the rotating pins.

To assemble: Fit the footrests on the rotary pins, placing the right footrest on the right-hand side and the left footrest on the left-hand side. Turn the footrests towards the centre of the chair until they come to a stop, whereby they will be automatically secured to the chair. Refasten the calf strap to its fixture.

2. ARMRESTS (FIG 11, PAG 2)

To loosen: Press button (A) and pull armrest upwards while maintaining button depressed. In order to facilitate removal, pull the front part of the armrest upwards.

To assemble: Place the armrest on its anchoring devices (front part first). Press the button (A) and push the armrest downwards. The armrest plate acts as a clothing guard against the wheel. As all these parts are secured by screws, they can easily be replaced when necessary or when certain accessories have to be adapted.

Warning: Lock armrests by securing release latch mechanism before using the wheelchair. Never lift the wheelchair by the armrests or footrests as these parts are detachable and it may cause damage to the chair and/or injury to the user.

3. BACKRESTS (Refs ; 3002, 3003, 3018, 3013, 3004) (fig. 12, pag.2)

To assemble: Insert the guide tubes (A) of the backrest in the rear guide tubes (B) of the seat's structure. Make sure that they are fully inserted. Tighten screws M8 (C) until the whole unit is properly anchored, and avoid removing the backrest in the future in order to prevent accidents.

To remove: Loosen the screws (C) until the whole unit is loose and can be removed. Remove the backrest by pulling it outwards until the guide tubes are free (A).

4. HEADREST

To remove: Press the button (A) and pull the headrest upwards. Do this one side at a time.

To install: Insert the headrest tube in the backrest tube. Press the button (A) and push the headrest tube downwards. Turn the tube until the button (A) fits into the lockhole. Do each side one at a time.

ADJUSTMENTS

1. HEIGHT OF THE FOOTPLATE. (FIG. 2, PAG. 1)

Loosen the screw in the extension tube with a n°10 spanner. Introduce the extension tube into the hanger until the desired position and reposition the screw passing through both tubes holes. Tighten the screw with the n°10 spanner.

2. WHEEL LOCKS. (FIG.3, PAG 1)

If the wheels are pneumatic, check for sufficient pressure. Loosen the locknut (A) with a n°13. Move the wheel lock towards the wheel if a stronger braking action is required and move it away from the wheel for less braking action. Retighten locknut (A) properly with the same. Check that the wheel turns freely when the lock is not being applied but becomes fully blocked when the lock is applied.

TROUBLESHOOTING AND MAINTENANCE

MAINTENANCE:

In addition to the maintenance and checks, general cleaning of both frame and upholstery can be undertaken. To maintain its appearance the upholstery should be cleaned regularly. Light soiling can be removed by sponging with a mild detergent and warm water, wiping in the direction of the texture. Rinse off with clean water before allowing to dry. Do not put upholstery in a dryer. If dirt is ingrained, the surface should be scrubbed with a soft brush. The frame should be cleaned regularly with a non-abrasive cleaner followed by polishing with a non-abrasive automotive wax.

CHECK LIST

A periodic check of the following procedures will help sustaining the life and efficiency of your wheelchair.

Weekly:

Check de pressure of the wheels following the instructions indicated on them. Check that the cross brace's central screw is tight and in good condition. Clean the upholstery with soapy water or adequate products. Clean the metallic parts of the chair with a dry cloth or with special products for chromium plated or painted surfaces. Check the wheel locks and adjust if necessary. Check wear of front and rear tyres.

Quarterly

Check and adjust if necessary: Castor play. Play in the castor forks. Push wheels' play. Offcentering of the push wheels. Lubricate with industrial oil or grease all parts subject to friction or wear. Retighten screws and nuts. A general revision of the chair's condition, repairing whatever is necessary.

Yearly : We recommend a complete revision once a year in a specialized establishment

SPECIFICATIONS

FOOTRESTS: 67° Swing away, detachable and height adjusting except in models 4007,4011,4002). Elevating leg-rests optional except in models 4001 and 4031 where they are std.

ARMRESTS: Detachable in all models except in 3004 (flip back armrests) and 3005 (Fixed)

STANDARD CASTORS: Folding models: 200 mm polyurethane (pneumatic optional) Fixed models: 125 mm polyurethane

STANDARD REAR WHEELS: 600mm polyurethane in models: 4007,4005,4001, 4021,4002 3003 and 3013
500mm polyurethane in models: 4020 . 317mm polyurethane in models: 4012,4031,4041,4010,4011.125mm polyurethane in models: 3004,3002, and 3018 Available many options with pneumatic tyres.

WHEEL LOCKS: high push-to-lock; options: extension handles and drum brake

SPARE PARTS: Each wheelchair component is available as an spare part.

ACCESORIES : Armrests: Long padded, desk armrest and height adjusting. Foot-rests: elevating. Wheels: Pneumatic tyres, quick release, hemiplegia system, and spoked or mag wheels. Wheel locks: extension handles and drum brake. Others: belt, anti-tip tubes, table, crutch and dripping device supports.

STANDARD MEASURES

Model	Seat width min-max	Seat height	Seat depth	Backrest height	Overall width min-max	Overall length	Standard weight
4007	36-48	51	43	43	56-68	107	17,5 Kg
4005	36-48	51	43	43	56-68	107	18,5
4015	36-48	51	43	43	56-68	107	18,5
4020	32-36	46	36	38	54-58	88	17
4001	36-48	51	43	43	56-68	121	25
4021	36-48	51	43	43	56-68	106	20
4031	36-48	51	43	43	52-64	111	24,5
4041	36-48	51	43	43	52-64	106	19,5
4010	36-48	51	43	43	52-64	102	18
4012	36-48	51	43	43	52-64	102	17
4011	36-48	51	43	43	52-64	101	17
4002	36-48	51	43	43	56-68	92	18
3004	45	53	44	40	53	88	13
3002	43	48	45	50	53	100	17,5
3003	43	48	45	50	65	114	21
3018	43	48	45	50	53	106	21
3013	43	48	45	50	65	114	22,5

GUARANTEE

The frame and all manufactured and distributed parts and components including frame components, axle plates, castor, plates, armrests, backrests, etc. are guaranteed against defects in materials and workmanship for one year from date of purchase. Those items not guaranteed are tyres and tubes, upholstery, armrests pads and push handle grips.

SUNRISE MEDICAL will repair or replace parts as required, provided that an authorisation for such return has been acquired by an established SUNRISE MEDICAL supplier and that the part or parts in question are returned within the prescribed period of guarantee.

Labour charge incurred in the installation or repair of the above mentioned parts plus all freight charges are not included in the guarantee.

SUNRISE MEDICAL will not guarantee any parts damaged due to improper installation and or repair on the part of the individual owner, neglect, accident or misuse. Parts altered or serviced by an agency not authorised by SUNRISE MEDICAL are not warranted.

The manufacturer reserves the right to alter without notice any weights, measurements, or other technical data shown in this manual.

All figures, measurements, and capacities shown in this manual are approximate and do not constitute specifications.

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